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Additional elements less than 1%: Mn, Ni

Contains no nickel, beryllium or iron

Technical data

Yield strength*	R _{p0.2}	700 MPa
Tensile strength*	R _m	900 MPa
Hardness*	HV10	340
Fracture elongation*	A ₅	7 %
Modulus of elasticity*	E	200 GPa
Density		8.6 g/cm ³
Solidus		1290 °C / 2354 °F
Liquidus		1415 °C / 2579 °F
CTE (25 – 500 °C / 77 – 932 °F)		14 x 10 ⁻⁶ K ⁻¹
Color		white
Weight per ingot		approx. 6 g

* Variations of +/- 10% are possible depending on the batch.

Use	CoCrMoW bonding alloy compatible with conventional ceramic materials with appropriate CTE range
Melting	Torch, Induction (HF), Light arc
Mirror like melt surface (HF)	Yes
Exact casting moment (HF)	As soon as the oxide layer starts to tear up
Pressure casting	250 – 450 mbar Observe instructions for use REF 989-448-00!
Welding	Possible using methods such as laser welding and TIG impulse welding
Filler material	CoCr welding wire, REF 528-215-10 (0.25 mm) REF 528-210-10 (0.35 mm) REF 528-200-10 (0.5 mm)
Presoldering	CoCrMo-Sold 1, REF 102-306-00
Flux	rema [®] -Flux 1, REF 102-304-00
Recommended ceramic	Ceramics with suitable CTE range such as Carmen [®] and CCS, from Dentaurum
Cooling of ceramics in the furnace	Slow cooling, unless otherwise recommended by the ceramic manufacturer

For further information see:

remanium[®], non-precious metal alloys for crown and bridgework,
instructions for use **REF 989-448-00**

Available free of charge from Dentaurum or on the internet at
www.dentaurum.com

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