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## Instructions for use Discovery®-Brackets

EN

### Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Our products are regularly modernized to the latest technical development. For this reason we advise examining the most up-to-date operating instructions supplied for any possible changes, even if the product is frequently used.

## Directions

### 1. Tooth preparation

Check each tooth carefully for any existing damage to the enamel to ensure there are no "surprises" with tooth damage during debonding.

Adhere to the bracket adhesive manufacturer's instructions for conditioning the teeth.

### 2. Selecting the bracket adhesive

Discovery® brackets can be retained using any standard bracket adhesive. We recommend our ConTec, NO-MIX and MIX adhesives or SmartBond®. Please follow manufacturer's instructions for bracket adhesive being used.

### 3. Removal and handling

Grip the Discovery® bracket with haemostatic forceps (e.g. REF 025-277-00) mesially and distally or on the distal or mesial bracket wings to remove the bracket from the packaging and follow the working stages below to ensure correct use. Do not handle the patented laser-textured base of the Discovery® bracket or contaminate it in any way, as this could detrimentally affect the bond.

### 4. Applying the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply only a small amount to the centre of the bracket base, but enough to cover the whole bracket base. **This guarantees an optimum bond.**

### 5. Application of bracket

After applying the adhesive, position the Discovery® bracket on the tooth in the usual way. Align the Discovery® bracket optimally using the angulation aid on the haemostatic forceps and press it onto the tooth so that any excess adhesive exudes from the side of the bracket base.

Ensure that there is not any contact with the opposing tooth to prevent the risk of abrasion and/or premature bracket loss. If necessary, raise the bite with a bite-raising appliance.

After having placed the bracket, immediately remove all access adhesive **completely** from around the bracket before it hardens. **This is of special importance to ensure easy debonding at the end of the treatment period.**

### 6. Ligating the arches

Please observe setting times of the adhesive carefully according to the manufacturer's instructions. When adhesive has hardened, ligate as usual with wire or plastic ligatures.

### 7. Bracket debonding

Allow for the excellent bonding characteristics of Discovery® brackets when debonding:


Discovery® brackets can be debonded using Weingart pliers, e.g. REF 003-120-00. Grip the bracket mesially and distally or diagonally at the wings, close the pliers gently and using the debonding channel of the Discovery® bracket loosen the base carefully from the tooth. Then twist the bracket clockwise or anticlockwise to remove it from the tooth without pulling.

### Additional facts

- Please review the technical information with reference to torque, angulation, size, etc. on the product label or in our catalog. Allergic reactions to Discovery® brackets are not known.
- Store Discovery® brackets in their original package.
- There are no particular regulations to dispose of Discovery® brackets after treatment.
- Should you have any questions, please contact your Dentaureum sales representative or our orthodontic fixed appliance hotline on +49 (0)7231 / 803-550.

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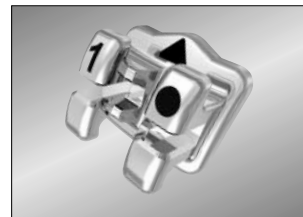EN

### Warranty

Dentaureum guarantees a faultless quality of the products manufactured by us. The information in these instructions for use is based upon our own experiences. The user himself is responsible for the correct processing or use of the products. We cannot be held responsible for failures, as we have no influence on the processing. However, any claim for compensation only applies to the commercial value of our products.

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